

Instructions For Use
Supplement:
S532 Series
DA165NP
DA135 CT/e

**Limits for Operation, Transport and
Storage**

Compatibility List



Table of Translations

Contents

English, English (US)	3
------------------------------------	----------

Replace Following List with Table of Translations:

- English
- Arabic
- Bulgarian
- Chinese (Simplified)
- Croatian
- Czech
- Danish
- Dutch
- Estonian
- Finnish
- French
- German
- Greek
- Hungarian
- Italian
- Japanese
- Korean
- Latvian
- Lithuanian
- Polish
- Portuguese
- Romanian
- Russian
- Slovak
- Slovenian
- Spanish
- Swedish
- Turkish

Instructions for Use Supplement

**Limits for Operation, Transport and
Storage**

Compatibility

English, English (US)

Limits for Operation, Transport and Storage

X-Ray Tube Assembly				
Environmental Limit	S532B	S532Q	DA165NP	DA135 CT/e
Operation				
Ambient Temperature [°C]				
Minimum	+15	+15	+5	+5
Maximum	+44	+44	+35	+35
Ambient Humidity [%]				
Minimum	N/A	N/A	30	30
Maximum	N/A	N/A	70	70
Transport and Storage				
Ambient Temperature [°C]				
Minimum	-20	-20	-20	-20
Maximum	+70	+70	+70	+70
Ambient Humidity [%]				
Minimum	15	15	5	30
Maximum	75	75	95	95
Ambient Atmospheric Pressure [kPa]				
Minimum	70	70	70	70
Maximum	106	106	106	106

Compatibility

This X-ray tube assembly is designed and manufactured to be compatible with specific diagnostic imaging equipment. Prior to installation, ensure that the X-ray tube assembly and X-ray imaging system are compatible. Installation of this device to a system not expressly identified as compatible, or modification of this device to facilitate such installation, will void all warranties and may result in increased safety risk.

The matrix below identifies compatible X-ray imaging systems with an “X”.

X-ray imaging systems which do not have an “X” or are not listed are not validated.

X-ray Imaging System	X-ray Tube Assembly			
	S532B	S532Q	DA165NP	DA135 CT/e
MX 8000 Dual/Quad	X			
Somatom Sensation 4	X			
Somatom Volume Zoom	X			
Somatom Volume Access	X			
Brilliance 6, 10, 16		X		
MX8000 IDT		X		
Sensation 10/16/Cardiac		X		
BrightSpeed™ Edge (Select/Lite)			X	
BrightSpeed™ Elite (Select/Lite)			X	
BrightSpeed™ Excel (Select/Lite)			X	
HiSpeed DX/i			X	
HiSpeed FX/i			X	
HiSpeed LX/i			X	
Optima™ CT520			X	
Optima™ CT520 Pro			X	
Brivo CT315				X
Brivo CT325				X
Brivo CT385				X
HiSpeed CT/e				X
HiSpeed CT/e Dual™				X